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DE

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Instructions for use

EN

Thank you for choosing a product from Rehaforum.

Please read the complete instructions for use carefully before installing, adjusting or using the product for the first time. This will provide you with important information that will allow you to fully utilise the technical advantages. You will also receive information that will help you to ensure operational safety and maximise the value of your product. If you require further information about the product, please contact your specialist dealer.

These instructions for use contain all the necessary information for handling the product. Please keep them in a safe place so that they are available for future reference.

Product description

The RFM bedside table provides support when reading, eating or drinking at the bedside or armchair. The four castors make it easy to move from the bed to another room. The steel construction provides a stable stand, which is reinforced by the two locking brakes on the rear castors. The table top is non-slip, easy to clean and adjustable in both angle and height.

Scope of delivery

Bed table RFM, not pre-assembled

Instructions for use

Technical data

Weight: 7.5 kg

Width: 61 cm

Depth: 45 cm

Height: 65 cm - 94 cm

Table top dimensions: 62 cm x 41 cm

Maximum load: 18 kg

Material: lacquered steel, plastic

Assembly

1. The RFM bed table is delivered partially pre-assembled in a cardboard box. Check the contents and packaging for integrity and completeness.

You will receive the following individual parts: 1x lower frame, 1x upper frame with table top, 2x wheel without brake, 2x wheel with parking brake, 1x locking screw.

2. Remove the lower frame and place it upside down on the floor. Now attach the four wheels. Make sure that the wheels with the brakes are at the rear end of the frame. Turn the frame over again.

3. Align the upper frame with the tabletop and secure it to the lower frame using the black locking screw.

4. To adjust the inclination of the tabletop, loosen the red clamping screw underneath the tabletop. This loosens the teeth. Set the desired inclination and then tighten the clamping screw again. Make sure that the teeth close again.

5. The height of the bed table can be adjusted using the knurled screw at the bottom of the frame.

6. Always lock the brakes on the two rear wheels to ensure a secure stand. Only release these if the RFM bed table is to be moved.

Safety Instructions

- Check the RFM bed table for damage before each use. Do not use a defective bed table!
- Before each use, check that the bed table is stable and that the brakes are locked.
- Before each use, make sure that the locking screw and the knurled screw are tightened.
- Only use the bed table indoors on a level surface.
- Do not lean your weight on the tabletop and do not pull yourself up by it. There is a risk of tipping.
- The maximum load capacity of the tabletop is 18 kg.

Cleaning and disinfection

The product should be cleaned and disinfected regularly. To clean the product, rinse it with warm water and use a mild soapy solution for cleaning. Dry the product with a soft cloth and ensure good ventilation. Disinfection can be carried out using a commercially available disinfectant. Follow the manufacturer's instructions for use. Do not use abrasive or corrosive cleaning agents or high-pressure or steam cleaners!

Maintenance

If you notice any damage, signs of wear and/or loosening of the mounted parts, contact your specialist dealer.

Disposal

Disposal options may vary depending on local regulations. Check with your local waste disposal centre whether the product can be disposed of as normal household waste or whether there are separate regulations for disposal.

Warranty

The warranty period is 24 months and applies to all product defects that are demonstrably attributable to manufacturing or material defects. Excluded from the warranty are Damage caused by natural wear and tear or intent. Furthermore, the warranty does not cover damage caused by incorrect cleaning and care products or damage caused by negligent and improper operation or use.

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn